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20000006235

KAN TORBASI ÇİFTLİ, CPDA-1 450 ML

TANIM

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
20000006235 BLOOD BAG DOUBLE, CPDA-1 450 ML

DESCRIPTION

Closed blood bag systems made of PVC, consisting of 16 g needle, transfer bags and including CPDA-1 as the anticoagulant solution.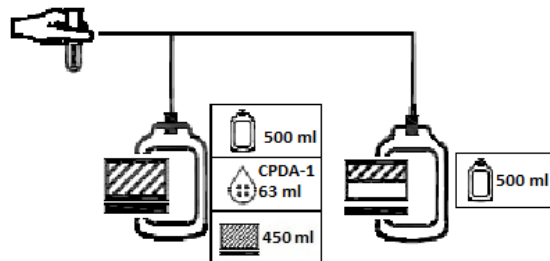

INTENDED USE

For collection, separation, transporting, processing and storage of red blood cell (RBC) suspensions and plasma per "Guide to the Preparation, Use and Quality Assurance of Blood Components".

INTENDED USER AND INTENDED USE ENVIRONMENT

This product is intended to be used by certified Blood Bank personnel in the Blood Bank.

INDICATIONS FOR USE

This product is intended to be used for collection of whole blood and preparing blood components from whole blood (plasma and red blood cells) for all population to aid any treatment. Blood components can be used for restoring blood components, improving oxygen delivery to tissues and restoring clotting factors according to the treatment decision of physician.

CONTRAINDICATIONS

This product is not intended to be reused or re-sterilized. The risks that may arise from reuse or re-sterilization (e.g. infection, cross contamination, loss of performance) could result in serious injury or death.

PRECAUTIONS

- Do not use if needle protector is opened.
- Do not add medication to blood.
- Cross match the contents of the bag with recipient before transfusion.
- Use transfusion sets in compliance with EN ISO 1135-4.
- Follow validated blood component preparation procedures per local requirements.
- The tubes are suitable to be connected with a Sterile Connecting Device.
- It is intended for human blood and components.
- During shipment, up to two shipping boxes may be carried at the same time manually or on a trolley, they must not be dropped or any heavy items loaded on top of them, and, they should be loaded in such a way to prevent them from falling over.
- Single use only, after usage it should be thrown as a Medical Waste. **Caution:** Risk of infection when used more than once.
- For the explanation of symbols, see Symbols Description Table.

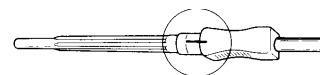
Figure 1

Caution: This medical device contains di(2-ethylhexyl) phthalate (DEHP). Children including developing foetus, premature neonates and neonates and therefore potentially pregnant and nursing women are at higher risk to DEHP exposure, although humans have been observed to be less sensitive than rodents. Furthermore, exchange transfusion of blood in neonates, extracorporeal membrane oxygenation treatment of neonates and adults, total parenteral nutrition in neonates, haemodialysis, enteral nutrition in neonates and adults, heart transplantation or coronary artery bypass graft surgery, massive blood transfusion of red blood cells and plasma or peritoneal dialysis may lead to high exposure to DEHP. Medical procedures should not be avoided however, as the benefits outweigh any possible health risks associated with DEHP exposure. Please refer to current literature to make an informed decision.

GENERAL PREPARATIONS BEFORE USAGE

- Open the outer wrap at the tear notches and remove one blood bag system, check for any visible defects of tubes, needle, anticoagulant and additive solutions in terms of appropriate volume, discoloration or particulate contaminants. **Caution:** Do not use defective sets, make sure the solutions are clear.
- Apply blood pressure cuff or tourniquet on donor's arm, identify venipuncture site and release the cuff or tourniquet.
- Disinfect the venipuncture site. Cover the area with sterilized gauze strip and do not touch until the time of venipuncture. **Caution:** Beware of iodine sensitive donors!
- Signs on the needle protector and hub should be in the same line (Figure 1). Apply some pressure to determine any leakages.
- Place the collection bag below the level of donor's arm.

BLOOD COLLECTION (indicative)

- Reapply tourniquet or inflate blood pressure cuff until previously selected vein is prominent again.
- Hold the needle from marked side. To remove the needle cap, twist to break the seal between the needle and its cap and remove it by sliding it down the axis of the needle. **Caution:** Be sure not to touch the cannula!
- Apply the venipuncture immediately. Venipuncture is very important for a non-coagulated blood.
- Strap the needle to the donor's arm.
- Mix blood and anticoagulant gently and periodically (approximately in every 45 seconds) during collection. If a mixer or a collection system is used, please refer to their operator's manual.
- Make sure there is continuous blood flow. If there is continuous, adequate blood flow and constant agitation, blood collection is expected to be completed under 12 minutes. **Caution:** If it is above 12 minutes, collected blood may not be suitable for Platelets, Fresh Frozen Plasma or Cryoprecipitate AHF.

Figure 2

7. Collect the quantity of blood within the limits indicated on the bag label. Monitor the blood being collected.
8. After blood collection is completed, stop the blood mixing device and loosen the tourniquet or deflate the pressure cuff.
9. Close the blood taking tube by tube sealing device and remove the needle from the system. **Caution:** Throw the needle kit to the medical waste.
10. Strip donor tubing as much as possible through the bag and start sealing. Work quickly to prevent the blood from clotting in the tubing. Invert bag several times to mix thoroughly, then allow tubing to refill with anticoagulated blood from the bag. Repeat this procedure twice (Figure 2).
11. Check the blood bag again for any faults.
12. Make sure all donor information required is filled out on the label on the blood bag.
13. After collection, whole blood should be placed at controlled room temperature (T) ($20 < T < 24$ °C) for a min of 2 hours and a max of 6 hours with 45° angle, if more or overnight holding period is used it can be stored at 1-6 °C in a blood bank refrigerator. For gaining fresh frozen plasma and platelets, complete all the steps within 8 hours after collection.

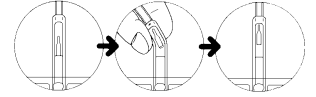
Figure 3

COMPONENT SEPARATION (indicative)

1. Load the whole blood filled bag system in the centrifuge buckets. **Caution:** Proper positioning of the bags will prevent burst of the bags during centrifugation.
2. Centrifuge at 2500 g for 15 minutes under +4° C. **Caution:** Centrifuge settings may vary according to the centrifuge brand used.
3. Place the centrifuged bag to the blood component extractor.
4. Break the breakaway part of main bag by turning it left and right (Figure 3).
5. Transfer the plasma into the transfer bag, and seal the tubing.

STORAGE

1. Store the plasma in the deep freeze. If the separation takes place within 6 hours after collection, it can be stored at -18 °C/-25 °C for 3 months, -30 °C/-40 °C for 6 months and at lower temperatures for 12 months.
2. RBCs should be placed into blood bank refrigerators designed for this process under 1-6 °C.

TRANSFUSION

1. Inspect the container and blood for any defects.
2. Before transfusing the blood, follow the validated local procedures.
3. Before transfusing the blood, make sure that serological test results are acceptable, cross match test results match with the recipient. If appropriate, apply cross match again at bedside.
4. Hold the top part of the T part with one hand and bag from the bottom part of T part with other hand, and twist of the upper part by 270° (Figure 4).
5. Keep holding the bag from T part, remove the cap of the transfusion spike with other hand.
6. Insert the spike by ¾ of its length into outlet port. Do not touch the spike of the transfusion set and open outlet port to prevent contamination.
7. After inserting the spike, squeeze the bag to the level that transfusion set filter is filled with blood.
8. Hang the bag from its holder. The unit is ready for transfusion.

Figure 4

SYMBOL	TÜRKÇE	ENGLISH	FRANCAIS	ITALIANO	DEUTSCH	POLSKI	ESPAÑOL	PORTUGUÊS	DANSK	SVENSKA	ΕΛΛΗΝΙΚΑ	BULGARIAN	SLOVENSKY
	XX mL Kan ya da Kan Bileşeni Kabi + nominal hacim	Blood or blood component container + nominal volume	Réceptacle contenant du sang ou des composants sanguins + volume nominal	Sacca sangue o emcomponente + volume nominale	Behälter für Blut oder Blutkomponenten + Nennvolumen	Pojemnik na krew lub składniki krwi + objętość nominalna	Bolsa de sangre o hemocomponente + volumen nominal	Saco de sangue ou componente de sangue + volume nominal	Blod- eller blodkomponentbeholder + nominelt volum	Blodkomponent-påse + nominellt volym	Περιέκτη αίματος ή συστατικού αίματος + ονομαστικός όγκος	Контейнер за човешка кръв или кръвни съставки + номинален обем	Vak na krv alebo zložku krvi + nominálny objem
	Tromboosit	Platelets	Plaquettes	Plastrine	Thrombozyten	Płytki krwi	Plaquetas	Plaquetas	Trombocytter	Trombocytter	Λιπιδεσθία	Тромбоцити	Krvné doštičky
	Tam kan bileşeni kabi	Whole blood	Sang total	Sangue intero	Vollblut	Krew	Sangre total	Sangue total	Fuldblod	Helblod	Περιέκτη ολικού αίματος	Цяла кръв	Celá krv
	Kırmızı kan hücreleri konsantresi	Red blood cell concentrate	Erythrocytes	Emazie concentrată	Erythrozyten	Erythrocyty	Hematies	Eritrocitos	Erytrocytter	Erytrocyter	Συμπυκνωμα Ερυθροκυττάρων	Еритроцитен концентрат	Koncentrát červených krviniek
	Buffy Coat	Buffy coat	Couche leucoplaquettaire	Buffy Coat	Buffy-Coat Beutel	Kozuszek leukocytarno-płytkowym	Capa leucoplaquetária	Camada leucoplaquetária	Buffycoat	Buffy Coat	Buffy coat	Кръвен серум	Buffy-coat
	Tromboosit kabi	Platelets container	Poche de conservation du concentré plaquettaire	Sacca per plastrine	Thrombozyten Beutel	Pojemnik z krwinkami płytkowymi	Bolsa para Plaquetas	Recipiente de plaquetas	Trombocytbeværin gæpose	Trombocyt påse	Περιέκτη αιμοπεθίδων	Сак за съхранение на тромбоцитен концентрат	Vak na krvné doštičky
	Kırmızı kan hücreleri kabi	Red blood cell container	Poche contenant les Erythrocytes	Sacca per la conservazione delle emazie concentrate	Erythrozyten Bautel	Pojemnik czerwonych krwinek	Bolsa para Hematies	Recipiente de ematies	Pose til Erytrocytter	Erytrocyter påse	Περιέκτη ερυθροκυττάρων	Сак за съхранение на еритроцитен концентрат	Vak na červené krvinky
	Plazma kabi	Plasma container	Poche contenant le plasma	Sacca per la conservazione del plasma	Plasma Beutel	Plasma container	Bolsa para Plasma	Recipiente de plasma	Pose til plasma	Plasma påse	Περιέκτη πλάσματος	Сак за съхранение на плазма	Vak na plazmu
	Buffy Coat kabi	Buffy coat container	Poche contenant la couche leucoplaquettaire	Sacca per buffy coat	Buffy-Coat Beutel	Pojemnik z kożuskiem leukocytarno-płytkowym	Bolsa para "Buffy coat"	Recipiente de Camada leucoplaquetária	Pose til buffycoat	Buffy coat påse	Περιέκτη Buffy coat	Сак за съхранение на кръвен серум	Vak na buffy-coat
	Tam kan bileşeni kabi	Whole blood container	Poche de sang total	Sacca per sangue intero	Vollblutbeutel	Cały worek krwi	Contenedor de sangre total	Recipiente de sangue total	Pose til fuldblod	Helblodspåse	Περιέκτη ολικού αίματος	Сак за съхранение на цяла кръв	Vak na celú krv
	Işlem kabi	Processing container	Poche de transfert	Sacca di trasferimento	Transfer Beutel	Pojemnik do przetwarzania	Bolsa de transferencia	Recipiente de transferencia	Transferpose	Överföringspåse	Περιέκτη επεξεργασίας	Контейнер за преработка	Vak na spracovanie
	Antikoagulan sölüsyon	Anticoagulant solution	Solution anticoagulante	Soluzione anticoagulante	Antikoagulans-Lösung	Rozwiązanie przeciwzakrzepowe	Solución anticoagulante	Solução anticoagulante	Antikoagulerende løsning	Antikoagulanlösning	αντιπηκτικό διάλυμα	Антикоагуляционен разтвор	Antikoagulačný roztok
	Katkı çözeltisi	Additive solution	Solution additive	Soluzione conservante	Additivlösung	Rozwiązanie dodatk	Solución aditiva	Solução aditiva	Additiv løsning	Tillsatslösning	Προσθετο διάλυμα	Добавен разтвор	Přídavný roztok
	Lökosit filtreleme	Leucocyte filtration	Filtration des leucocytes	Filtrazione leucocitaria	Leukozytenfiltration	Filtracja leukocytów	Filtración de leucocitos	Filtração de leucócitos	Leukocytffiltrering	Leukocytffiltrering	Διήθηση των Λευκοκυττάρων	Филтрация на лейкоцити	Filtrácia leukocytov
	Diş ambalajda göze görülen bozuma belirtilir varsa kullanmayınız.	Do not use if there is visible sign of deterioration	Ne pas utiliser si des signes de détérioration sont visibles	Non utilizzare in caso di segni visibili di deterioramento	Nicht verwenden, wenn Anzeichen des Verfalls sichtbar sind.	Nie używać, jeśli nosisi ślady uszkodzenia	No utilizar si hay signos visibles de deterioro	Não utilizar se forem visíveis sinais de deterioração	Må ikke anvendes, hvis der er synlige tegn på nedbrydning	Använd ej vid synliga tecken på försämring	Μην το χρησιμοποιείτε εάν υπάρχει ορατή ένδειξη αλλοίωσης	Да не се използва, ако има видими белези на увреждане	Nepoužívať pri viditeľných známkach poškodenia
	Doğal kauçuk lateks içermeyin.	Free of natural rubber latex	Dépourvu de latex naturel	Privò di lattice naturale	Latexfrei	Nie zawiera naturalnego lateksu	Libre de látex de caucho natural	Não contém látex de borracha natural	Frit for naturgumlatex	Latexfri	Δεν περιέχει φυσικό λάτεξ	Не съдържа латекс	Neobsahuje prírodný kaučukový latex
	DEHP içerir ya da bannır.	Contains or presence of DEHP	Trace ou présence de phtalates : phtalate de di-2-éthylhexyle (DEHP)	Contiene ftalati o presenta tracce di ftalati: Di-(2-ethyl)ftalato (DEHP)	Enthält DEHP	Zawiera ftalan	Contiene o puede que haya ftalatos presentes Di-(2-ethyl)ftalato (DEHP)	Presença ou conteúdo de DEHP	Produkt eller emballage indeholder DEHP	Innehåller eller förskott av DEHP	Περιέχει φυσικό DEHP ή ftalyl	Съдържа или наличие на DEHP	Obsahuje alebo môže byť prítomný DEHP
	Hava vermeyiniz.	Do not vent	Ne pas utiliser de prise d'air.	Non introdurre aria	Nicht belüften	Przechowywać szczelnie zamknięte	No ventilar	Não ventilar	Må ikke udluftes	Får ej ventileres	Μην χρησιμοποιείτε αεραγωγό	Да не се провие	Neodvzdovávať
	Radyasyon ile svi yolu sterilizasyon	Sterile fluid path by radiation	Trajet stérile. Stérilisé par irradiation.	Percurso del liquido sterile. Sterilizzato per irradiazione	Sterile Fluidweg durch Strahlung	Nie stosować. Jeśli opakowanie jest uszkodzone	Via de fluido estéril por RADIACION	Percurso do fluido esterilizado por radiação	Steril vaskbænde ved bestråling	Steril vâskbænda genom strålning	Αποστειρωμένη διαδρομή υγρού από ακτινοβολία	Стерилен поток чрез радиация	Dráha tekutiny je sterilizovaná ionizačnim žiarením
	Isi ile svi yolu sterilizasyon	Sterile fluid path by STEAM	Trajet de fluide stérilisé par VAPEUR	Percurso del liquido sterile. Sterilizzato per Ossido di etilene	Sterile Fluidweg durch Dampf	Sterylnie droga przepływu pynu przez parę	Via de fluido estéril por Vapor	Percurso do fluido esterilizado pelo vapor	Steril vaskbænde med damp	Steril vâskbænda genom ÅNGA	Αποστειρωμένη διαδρομή υγρού από τον ατμό	Стерилен поток чрез парна стерилизация	Dráha tekutiny je sterilizovaná parou
	Hasar görmüş yitirilmiş veya delinmiş paketi kullamayınız.	Do not use if package is damaged	Ne pas utiliser si le produit, sa barrière de stérilité ou son emballage a été endommagé ou s'il présente des signes d'altération.	Non utilizzare se il prodotto, la sua barriera sterile o il suo contenitori sono danneggiati o mostrano segni di deterioramento	Nicht verwenden, wenn die Verpackung beschädigt ist	Sterylnie droga przepływu pynu przez parę	No usar si el producto, su barrera estéril o su envase está dañado o muestra cualquier signo de deterioro	Må ikke anvendes, hvis produktet, sterilbarrieren eller pakningerne er beskadigede eller der er tegn på brud	Använd inte om förpackningen är skadad	Μην το χρησιμοποιείτε εάν η συσκευασία έχει υποστεί φθορά	Не използвайте, ако опаковката е с нарушена целост	Nepoužívať, ak je obal poškodený	
	Bu sıcaklık arası saklayınız.	Store between these temperature range	Stocker dans cette plage de température	Conservare in questo intervallo di tempo	Aufbewahren in diesem Temperaturbereich	Przechowywać w tym zakresie temperatur	Almacenar dentro de este rango de temperatura	Armazemar dentro deste intervalo de temperatura	Opbevarves ved dette temperaturområde	Lagras inom detta temperaturområde	Αποθηκεύεται μεταξύ των συγκεκριμένων θερμοκρασιών	Да се съхранява при следните температури	Skladujte v tomto teplotnom rozsahu
	Alüminyum paket açılışını takiben X gün içinde kullanınız.	Contents of Al pouch must be used within X days of opening	Le contenu de l'emballage doit être utilisé dans un délai de X jours après ouverture	Il contenuto della confezione di Al va utilizzato entro X giorni dall'apertura	Der Inhalt der Verpackung muss innerhalb von X Tagen nach dem Öffnen verwendet werden	Zawartość Al opakowania musi być zużyta w ciągu X dni od momentu otwarcia opakowania	El contenido de envases de aluminio debe utilizarse en los X días siguientes a su apertura	O conteúdo da embalagem de Al tem de ser utilizado no prazo de X dias após a sua abertura	Når Al pakningen er åbnet, skal indholdet anvendes inden for X dage	Innehållet i förpackningen måste användas inom X dagar efter öppnandet	Το περιεχόμενο της συσκευασίας αλουμινίου πρέπει να χρησιμοποιηθούν εντός Χ ημερών από το άνοιγμα	Съдържанието на опаковката да се използва до Х дни след отваряне	Obsah hliníkového foliového vaku sa musí spotrebovať do X dní od otvorenia.
	Paketi içeriği kullanmadan önce kullanınız.	Contents of pack must be used within 10 days of opening	Le contenu de l'emballage doit être utilisé dans un délai de 10 jours après ouverture	Il contenuto della confezione di Al va utilizzato entro 10 giorni dall'apertura	Der Inhalt der Verpackung muss innerhalb von 10 Tagen nach dem Öffnen verwendet werden	Zawartość opakowania musi być zużyta w ciągu 10 dni od momentu otwarcia opakowania	El contenido del envase debe utilizarse en los 10 días siguientes a su apertura	O conteúdo da embalagem deve ser utilizado no prazo de 10 dias após a sua abertura	Når pakningen er åbnet, skal indholdet anvendes inden for 10 dage	Innehållet i förpackningen måste användas inom 10 dagar efter öppnandet	Το περιεχόμενο της συσκευασίας πρέπει να χρησιμοποιηθούν εντός 10 ημερών από το άνοιγμα	Съдържанието на опаковката да се използва до 10 дни след отваряне	Obsah vrecka sa musí spotrebovať do 10 dní od otvorenia
	Aprojen svi yolu	Non-pyrogenic fluid path	Trajet apyrogène	Percurso del liquido apyrogeno	Nichtpyrogener Flüssigkeitspfad	Apirogena ścieżka przepływu	Paso de fluido apyrogeno	Percurso de fluido apyrogeno	Non pyrogen vaskbænde	Icke-pyrogen vâskbænda	Μη πυρογόνο διαδρομή υγρού	Непυрогенен поток	Dráha tekutiny je nepyrogenou
	Alet veya paket	pcs per package.	unités par emballage	pz per confezione	Stück pro Packung	Szt. w opakowaniu	Unidades que contiene el paquete	peças por pacote	stk pr pakke	st per förpackning	τεμάχια ανά ή συσκευασία	Брой в опаковка	ks v balení.
	Depolama işleminde 8 kolitten fazla üst üste konulmamalıdır.	Stacking limit by number. Maximum number of identical packages which may be stacked on one another, where "8" is the limiting number	Ne pas empiler plus de 8 cartons	Sovapporeni al max 8 carton/scatoie	Stapeln Sie nicht mehr als 8 Boxen	Dopuszczalna ilość warstw piętrzenia: "8"	Límite máximo de estiva es 8 cajas	Não empilhar mais de 8 caixas	Må ikke stables mere end 8 kasser	Stapla inte mer än 8 boxar	Μην στοιβάτε περισσότερο από 8 κιβώτια κατά την αποθήκευση	Складването в зависимост от броя продукти. Максимален брой пакети, които могат да бъдат подредени един върху друг, където "8" е определя максималния брой	Krajný počet pri stávaní. Maximálny počet rovnakých balení, ktoré sa môžu stávať jedno na druhé, pričom "8" je krajná hodnota
	Tek kullanılmıktır.	Do not re-use.	Ne pas réutiliser	Monouso	Nicht wiederverwenden	Nie używać ponownie.	No reutilizar	Não reutilizar	Må ikke genbruges	Ateranvänd inte.	Πρόν μιας χρήσης	Да не се използва повторно	Nevhodné na opätovné použitie.
	Kullanmadan önce kullanın kılavuzunu okuyunuz.	Read instructions for use.	Consulter la notice	Consultare le istruzioni per l'uso	Lesen Sie die Gebrauchsanweisung für die Anwendung	Przeczytaj instrukcję użycia	Leer las instrucciones de uso	Leia as instruções de utilização	Läs bruksanvisningen	Läs bruksanvisningen	Διαβάστε τις οδηγίες χρήσης πριν τη χρήση	Запознайте се с инструкцията за употреба	Prečítajte si pokyny na používanie.
	Dikkat.	Caution, read accompanying documents.	Attention, consulter les documents d'accompagnement.	Attenzione, consultare i documenti all'interno	Vorsicht, lesen Sie die Begleitdokumentation	Uwaga, przeczytaj dokumentację	Atención, leer los documentos adjuntos	Cuidado, leia os documentos de acompanhamento	Se brugsanvisningen	Varning, läs medföljande dokument	Προσοχή, διαβάστε τα συνοδευτικά έγγραφα	Внимание, прочетете придружаващите документи	Pozor, prečítajte si spríevodné dokumenty.
	Direkt güneş ışığı ve yüksek sıcaklığı maruz bırakmayınız.	Keep away from sunlight. Keep away from heat.	Protéger des rayons directs du soleil et d'une source puissante de chaleur.	Proteggere dalla luce solare diretta. Mantener lontano da fonti di calore	Vor Sonnenlicht schützen. Hitze fernhalten.	Chronicz przed nagrzaniem (cieplem)	Proteger del sol. Mantener lejos del calor	Mantênhá longe da luz solar. Manter afastado do calor	Beskyt mod direkte sollys og varme	Hållå borta från solljus. Hållå borta från värme	Να φυλάσσεται μακριά από το φως του ήλιου. Να φυλάσσεται μακριά από θερμότητα	Да не се излага на слънчева светлина. Да не се сержа	Chránit pred priamym svetlom. Chránit pred teplotou.
	Kuru yerde saklamayınız. Name maruz bırakmayınız.	Keep dry.	Conservar à l'abri de la pluie	Conservare in luogo asciutto	Trocken lagern	Chronicz przed wilgocią	Conservar en lugar seco	Manter seco	Opbevares tørt	Förvaras torrt	Διατηρήσει το στεγνό	Да се съхранява на сухо място	Składujte v suchu.
	Kolaylıkla kırılabilir, dikkatli tutunuz.	Fragile, handle with care	Fragile, manipuler avec précautions.	Facilmente divisibile, tenere ben ferma.	Verfalldatum	Szadno rozbielné, držte jı opatně	Fragil. Manipule con cuidado	Facilmente quebravel, segure-o com cuidado	Let i stykker, hold dem forsigtigt	Utgångsdatum	Ευθραστο, χειριστείτε με προσοχή	Чупливо, внимавайте при употреба	Krehké, zaobchádzajte opatrne
	Son kullanma tarihi.	Use by, date.	Date d'expiration	Scadenza	Vor scharfen Gegenständen entfernt halten	Nie używać po	Fecha de vencimiento	Data de validade	Utløbsdato	Hållå borta från vassa föremål	Ημερομηνία λήξης	Използвайте преди	Spotřebujte do datumu.
	Torbarli kesici ve delici materyaldir	Keep away from sharp objects.	Tenir à l'écart des objets tranchants	Tenere lontano da oggetti appuntiti o taglienti	Vor scharfen Gegenständen entfernt halten	Tryzmyć z dala od ostrzych przedmiotów	Mantener alejado de objetos cortantes	Manter longe de objetos cortantes	Holdes væk fra skarpe genstande	Förvaringspås for trombocytter upp till fem dagar	Να φυλάσσεται μακριά από αιχμηρά αντικείμενα	Παζете от остри предмети	Chránit pred ostrými predmetmi.
	5 güne kadar Tromboosit saklama torbasi	PLT storage bag up to five days	Poche de conservation de plaquettes jusqu'à cinq jours	Sacca per la conservazione delle plastrine fino a cinque giorni	Aufbewahrungsbeutel für Thrombozyten bis zu fünf Tage	Pojemnik do przechowywania płytek do pięciu dni	Bolsa para el almacenamiento de plaquetas hasta cinco días	Recipiente de armazenamento de plaquetas até cinco dias	Opbevaringspose for trombocytter op til fem dage	Förvaringspåse för trombocytter upp till sju dagar	Περιέκτη αποθήκευσης αιμοπεθίδων έως πέντε ημέρες	Сак за съхранение на тромбоцитен концентрат до пет дни	Vak na úschovu doštiček do piatich dní
	7 güne kadar Tromboosit saklama torbasi	PLT storage bag up to seven days	Poche de conservation de plaquettes jusqu'à sept jours	Sacca per la conservazione delle plastrine fino a sette giorni	Aufbewahrungsbeutel für Thrombozyten bis zu sieben Tage	Pojemnik do przechowywania płytek do siedmiu dni	Bolsa para el almacenamiento de plaquetas hasta siete días	Recipiente de armazenamento de plaquetas até sete dias	Opbevaringspose for trombocytter upp till sju dagar	Förvaringspåse för trombocytter upp till sju dagar	Περιέκτη αποθήκευσης αιμοπεθίδων έως επτά ημέρες	Сак за съхранение на тромбоцитен концентрат до седми дни	Vak na úschovu doštiček do siedmich dní
	Numune alma bölgesi	Sampling site	Site d'échantillonnage	Site di campionamento	Probenahmestelle	Punkt poboru próbek	Para toma de muestras	Local de amostragem	Prøvetagning sted	Prøvetagningställe	Σημείο δειγματοληψίας	сайт за вземане на проби	Miesto odberu
	Tekrar sterilize etmeyiniz.	Do not re-sterilize	Ne pas restériliser	Non risterrilizzare	Nicht erneut sterilisieren	Nie należy ponownie sterylizować	No reesterilizar	Não reesterilize	Må ikke reesteriliseres	Resteriliseres inte	Μην αποστείρυνετε ξανά	Не реставрирайте	Nesmie sa opätovne sterilizovať
	Üretici	Manufacturer	Fabricant	fabricante	Hersteller	Producent	Fabricante	Fabricante	Fabrikant	Tillverkare	Κατασκευαστής	Производител	Výrobca
	Seri Numarası	Batch Code	Code du lot	Codice batch	Batch-Code	Kod partii	Código de lote	Código de lote	Batchcode	Batchcod	Κωδικός Παρτίδας	Код на партидата	Číslo šarže
	Ürün Kodu	Catalogue Number	Numéro de catalogue	Numero di catalogo	Katalognummer	Numer katalogowy	Número de catálogo	Catálogo de número	Katalognummer	Katalognummer	Αριθμός Κατάλογου	Каталожен номер	Katálogovo číslo
	CE Uygulama İşareti ve Belgelendirme Kuruluş Numarası	CE Marking of Conformity and Identification Number of Notified Body	Marquage CE de conformité et numéro d'identification de l'organisme notifié	Marchio CE di conformità e numero d'identificazione dell'organismo notificato	CE- Konformitätsbescheinigung und Kennnummer der benannten Stelle	Oznaczenie CE zgodności i numeru identyfikacyjnego jednostki notyfikowanej	Marcação CE de conformidade e número de identificação do organismo notificado	Marcação CE de conformidade e número de identificação do Organismo Notificado	CE-mærkning af overensstemmelse og identifikation	CE-märkning av överensstämelse och identifikation	Επίσημο CE συμμόρφωσης και σφραγισμένο και σφραγισμένο του κοινοποιημένου οργανισμού	СЕ маркировка за съответствие и идентификационен номер на нотифициран орган	Označenie zhody CE a identifikačné číslo a notifikovaného orgánu